



ovesco
innovation in scope

► New



BougieCap

for the dilation of stenoses in the upper gastrointestinal tract

- Direct visual control during passage
- Easy and fast bougienage with fewer instrument change
- Eight BougieCap sizes, Ø 7–20 mm





**BOUGIE
CAP**



ovesco



Bougie Cap

A new method for the bougienage/dilation of stenoses and strictures in the upper gastrointestinal tract.

The BougieCap allows direct visual control during the bougienage. The tension on the tissue is visible while passing the stenosis with the cap, thereby avoiding overstretching.

Moreover, the procedure time is reduced due to less instrument change. The evaluation can be done directly after the bougienage thanks to the attached transparent cap.


Caps are available in 8 sizes (Ø 7-8-10-12-14-16-18-20 mm) and it will be attached onto the endoscope.

The distal end of the cap has three holes: two of them (lateral) for air insufflation and suction and one (center) for the insertion of a guide wire.


Application

Esophageal stenosis


Schematic

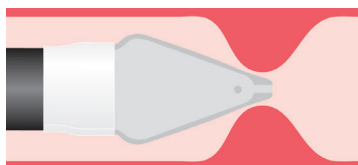

Guiding the endoscope with the attached BougieCap up to the stenosis.

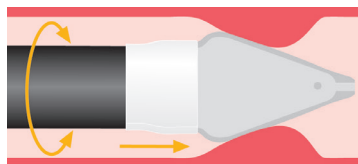


Carefully sliding the endoscope through the stenosis using gentle rotations under visual control.

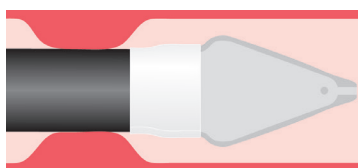

Successful passage of the scope through the stenosis.

Clinical example* (BougieCap used Ø 12–16 mm)

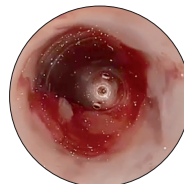

Ovesco Endoscopy AG
Dorfackerstr. 26
72074 Tuebingen/Germany

Phone +49 (0) 7071 96528-160
Fax +49 (0) 7071 96528-260
service@ovesco.com

Ovesco Endoscopy USA Inc.
120 Quade Dr
Cary, NC 27513
USA

Phone +1 919 651 9449
Fax +1 408 608 2077
customerservice@ovesco-usa.com

*Source: Prof. Meining, Universitätsklinikum Ulm, Endoskopie Live 2017 meets DGE-BV, Berlin



Ovesco Endoscopy AG is a medical device company specializing in the fields of flexible endoscopy and endoluminal surgery. Ovesco develops, manufactures, and markets innovative products for the treatment of gastrointestinal disease. Ovesco products stand for therapeutic efficacy and efficient application.

www.ovesco.com

English version 1 | 2017-10-04