

## 1. Prepare the required material

If LiftUp Kit was ordered (Art. Nr. 200.56.02; 1a):

LiftUp vial

+ 1 Injection needle from the kit (200.56.02)

+ 2 Syringes from the kit (200.56.02)

+ Spike or cannula

If LiftUp was ordered without accessory (Art. Nr. 200.56.01; 1b):

LiftUp vial

+ 23G Injection needle

+ Inflation device (min. 30 bar)

+ Spike or cannula

**Notes:** Take the LiftUp vial out of the refrigerator shortly before the procedure starts to support an easier injection.

2. Fill the injection needle with saline. Injection of an initial saline bubble (1 - 2 ml) can help to find the correct tissue layer.

3. Remove flip-off cap and pierce septum with a spike or cannula.




4. Draw up LiftUp in disposable syringe or inflation syringe.

5. Connect the disposable syringe or inflation syringe to the injection needle.

6. Do not close your hand around the filled syringe to prevent LiftUp from getting warm and become a gel in the syringe.

**Notes:** Especially during longer procedures, it can be helpful to rinse the injection needle with saline solution in between to prevent clogging.

**Tip:** Defoamer in the rinsing water can reduce the formation of bubbles when cutting through the LiftUp layer.

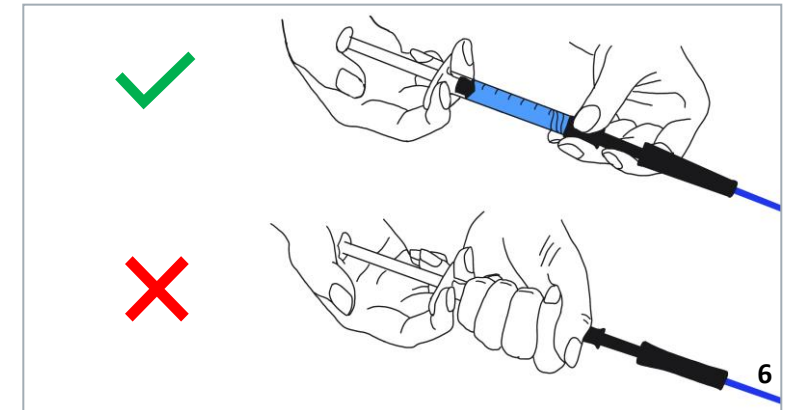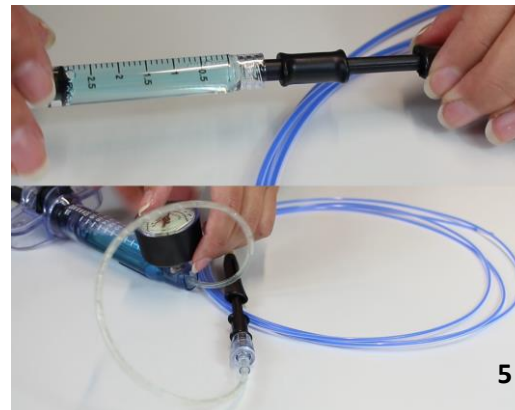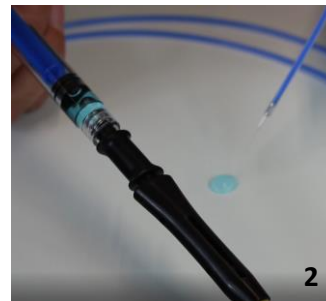

Scan the QR code,  
to watch the video instructions.

